
HEMATOCRIT CENTRIFUGE

Model : CDMH24

USER MANUAL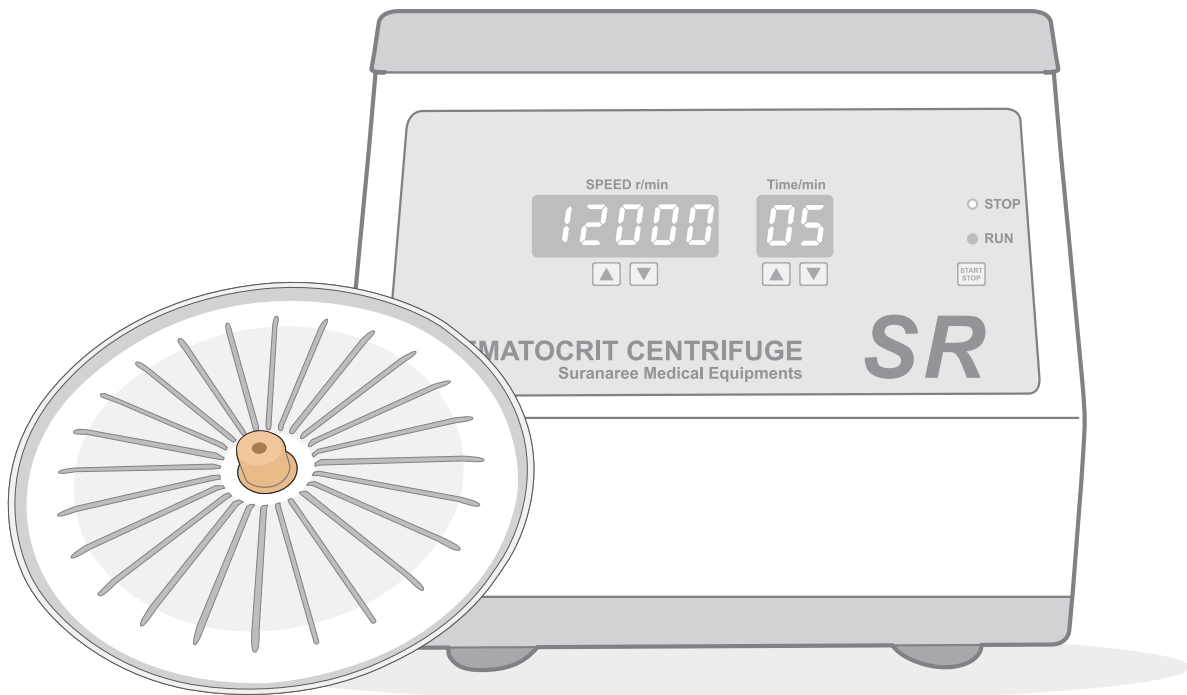

zest-med Co.,Ltd

896/1-2 OFFICE TOWER 1 SV CITY, 3FLOOR, RAMAIII RD., BANGPONGPANG
YANNAWA, BANGKOK 10120

T : +66 2682 9151-4 F : +66 2682 9155

Email : support@zest-med.com www.zest-med.com

TABLE OF CONTENTS


SECTION 1 DESCRIPTION AND INTENDED USE	3
SECTION 2 INSTALLATION	4
2.1 Unpacking	
2.2 Power Supply	
SECTION 3 SETUP PROCEDURE	4
3.1 Display Options	
3.2 Setup Procedure	
SECTION 4 OPERATING INSTRUCTIONS	4
4.1 Start Up	
4.2 Rotor Loading and Balancing	
4.3 Tube Centrifugation	
SECTION 5 MAINTENANCE	5
5.1 Cleaning	
5.2 Inspections	
5.3 Transportation	
SECTION 6 TROUBLESHOOTING	6
SECTION 7 WARRANTY	6
SECTION 8 SPECIFICATIONS	6

SECTION 1

DESCRIPTION AND INTENDED USE

Hematocrit Centrifuge Model CDMH24 is a lightweight centrifuge. The rotor will accommodate up to 24 capillary tubes. For convenience and ease of use, the Model CDMH24 also includes a built-in LED digital that displays the run time and speed (round per minute).

Hematocrit Centrifuge Model CDMH24 is intended to provide laboratory tests for hematocrit values (HCT) in the fields of biochemistry, microbic immunity genetics and blood separation, as well as general clinic tests.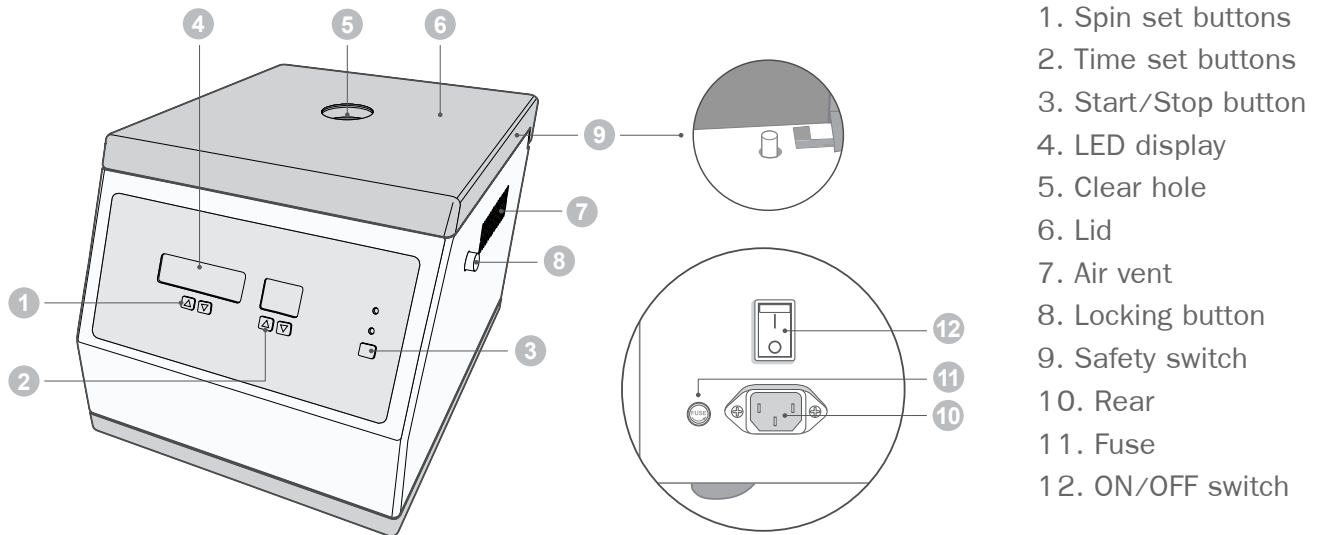


SECTION 2

INSTALLATION

2.1 Unpacking


Hematocrit Centrifuge model CDMH24 Shipping box contains: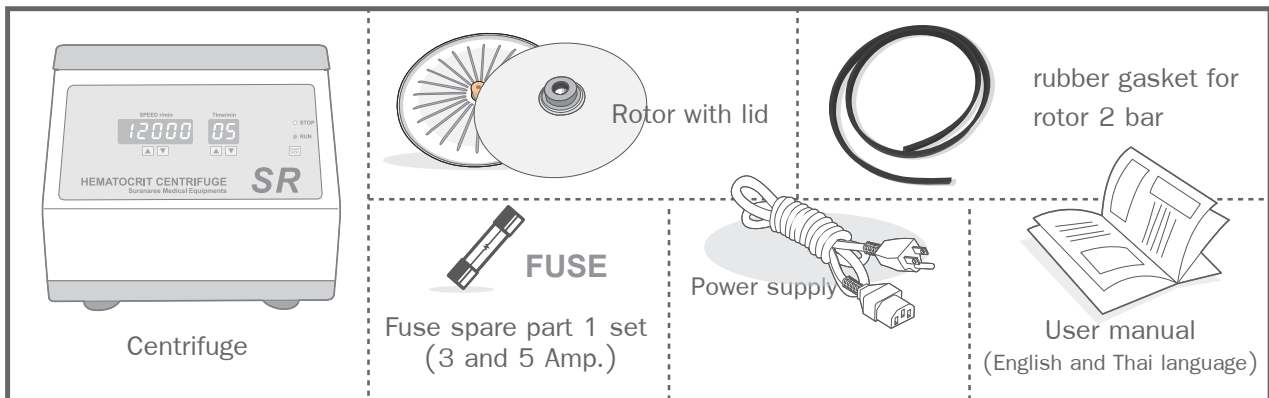

Read the user manual thoroughly before operating this system. Place the centrifuge on a convenient, level work surface along with the other items.

2.2 Power supply

Use ONLY the power supply packaged with the model CDMH24. Verify that the ON/OFF switch located on the back of the centrifuge is in the “OFF” position. Always plug the power supply into the rear of the device first and then plug the three prongs end into an electrical outlet. Press the ON/OFF switch to the “ON” position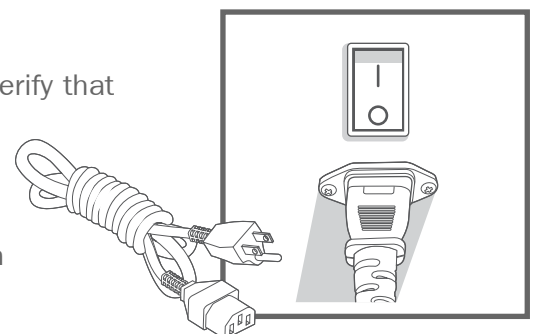


SECTION 3


SETUP PROCEDURE

3.1 Display options

This setup procedure will allow the user to store last functions in the display memory until a change is required. These functions include the selection of: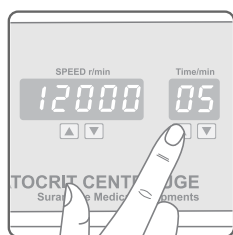

► Time 1–99 minute,
increment: 1 minute

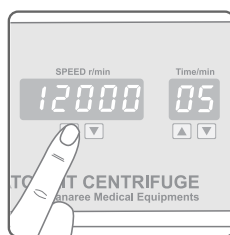► Speed 5,000–15,000 rpm,
increment: 100 rpm


3.2 Setup procedure

<p>SPEED r/min 12000 ▲ ▼</p>	Simultaneously press and release “▲”, “▼” button on “SPEED r/min” part for select the speed.	<p>Time/min 05 ▲ ▼</p>	Simultaneously press and release “▲”, “▼” button on “Time/min” part for select the run time.
<p>○ STOP ● RUN</p> <p>START STOP</p>	Press the “START/STOP” button to start running.	<p>● STOP ○ RUN</p> <p>START STOP</p>	The centrifuge will break and stop running if pressed during centrifugation.

SECTION 4

OPERATING INSTRUCTIONS

4.1 Start up

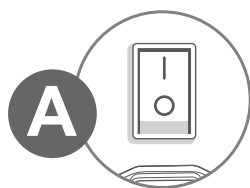Press the ON/OFF switch to the ON position.

The LED digital will display.

4.2 Rotor loading and balancing

Install the rotor. When an even number of capillary tubes (2, 4 or 6) are centrifuged, balancing is accomplished by placing the tubes on opposite sides of the rotor from one another so that the weight is distributed equally. If an odd number of tubes (1, 3 or 5) are centrifuged, use an empty capillary tube to balance the rotor. For smooth operation and extended life of the centrifuge, the rotor must always be balanced before the spin cycle is initiated.

4.3 Tube centrifugation


With the hematocrit tubes in place, close the cover. Start the run cycle by pressing the START/STOP button. The centrifuge will not operate unless the cover is closed and properly locked. Do not lean on the instrument.

Within seconds the centrifuge will reach the proper operating speed and the rpm will be displayed on the LED digital during the cycle. This number is a count of the completed spin cycles. The LED digital will display a countdown of time for the remainder of the run cycle, and the device will stop automatically.


When the run cycle is completed or START/STOP button is pressed, an automatic braking system will engage to completely stop the rotor. The cover lock will disengage to allow the cover to be opened. Open the cover and take out the rotor and samples.

SECTION 5 MAINTENANCE


5.1 Cleaning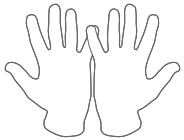


As with all electrical devices, make sure the centrifuge is unplugged before cleaning. Always wear protective clothing when using any cleaning materials.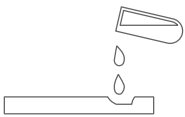

Never use bleach, abrasives or corrosive solvents. Do not spray or allow any liquid to get inside the centrifuge. Liquid will harm the electronics. Subsequent problems will not be covered under warranty.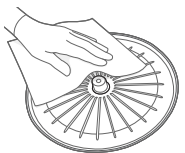


Use a disinfectant towel or a cloth slightly dampened with any non-corrosive disinfectant solution to clean the lid and other parts of the centrifuge housing. Dry all surfaces with a soft tissue or cloth after cleaning.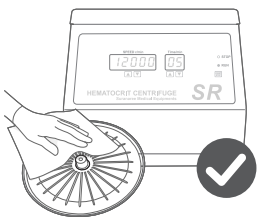


The rotor should be removed and cleaned at least monthly or as required by the laboratory protocol. Remove the rotor from the motor shaft by first unscrewing the rotor knob. Gently lift the rotor vertically off of the motor shaft. Make sure the rotor is thoroughly dry before reinstalling. Liquid left on the rotor will cause damage to the device. Re-install the rotor making certain that the rotor knob is tight.

5.2 Inspections

Periodically inspect the cover, cover gasket and rotor to ensure there are no cracks or damage.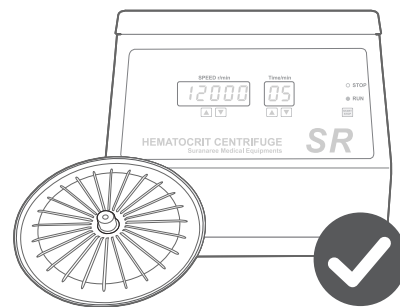

5.3 Transportation

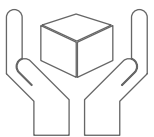When transporting the unit, place in secure packaging (preferably the original packaging material) which will prevent the unit from tipping over.

Do not drop the centrifuge, turn it upside down, or subject it to vibration.

SECTION 6

TROUBLESHOOTING

Problem	Possible cause	Solution
Can't turn centrifuge ON	<ol style="list-style-type: none"> 1. Blown fuse 2. Disconnected electrical cord 	<ol style="list-style-type: none"> 1. Contact your supplier to replace fuse (5A) 2. Connect electrical cord
Vibration	Unbalanced rotor	Balance rotor load
Broken test tubes	<ol style="list-style-type: none"> 1. Tubes are improperly positioned (unbalanced load) 2. Tubes are not placed all the way down 3. Rotor lid is tightened incorrectly 	<ol style="list-style-type: none"> 1. Balance load 2. Place tubes against rubber gasket on rotor 3. Ensure rotor lid is placed flat against rotor
Rotor will not spin	Lid not locked	Press firmly down on the lid tab.

SECTION 7


LIMITED WARRANTY

Your Hematocrit Centrifuge product is guaranteed to be free from defects in materials and workmanship for one (1) year under normal use from the date of purchase. This WARRANTY does not apply to any product damaged by misuse, abuse, neglect, improper line voltage, dropping, fire, water submersion or saturation. The warranty is void if non-authorized representatives service or dismantle the centrifuge. The warranty is void if damage occurs as a result of the lid being opened when the rotor is spinning. If a product is deemed to be under warranty, Zest-Med co., ltd. shall provide replacement of any defective part or product, labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to Zest-Med co., ltd.

SECTION 8

SPECIFICATIONS

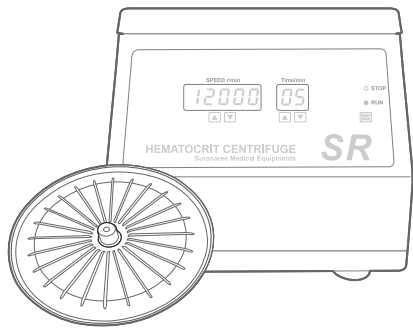Control system	Microprocessor Control
Maximum speed	15,000rpm(5,000-15,000rpm), increment: 100rpm
Timer	1-99minutes, increment: 1 minute
Rotor capacity	24 capillary tubes
Driving Motor	Brushless DC motor
Safety devices	Cover interlocking, Safety Switch, Switch
Power requirements	220V, 50/60Hz
Dimensions(mm)	250x325x240
Weight	8 kg

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